

Brief instructions for taking horsehairs

1. Rip strand of horsehairs from the mane or tail.
2. Horsehair has to include the hair bulb visible.
3. At least 20 bulbs are needed for laboratory analysis.
4. Place horsehairs directly in the labelled long envelope and seal it.
5. Mark the envelope with clear identification of the sample (eg. assigned a sample number or other identification).
6. Place the long envelope with horsehairs in the bubble envelope with the return address and send it by regular mail to the laboratory address:

Genomia s.r.o.
Teslova 3
301 00 Plzeň
Czech Republic
Europe