

Brief instructions for taking horsehairs

1. Mark a clean paper envelope / plastic bag with clear identification of the sample (eg. assigned a sample number or any other identification).
2. Pull out a strand of horsehairs from the mane or tail. Horsehairs has to have visible hair roots. At least 20 roots are needed for genetic analysis.
3. Place horsehairs directly in the marked envelope / plastic bag and close it.
4. Place the envelope / plastic bag with horsehairs in a post envelope and send it to the Genomia laboratory. Our address is:

Genomia
Republikánská 6
31200 Plzeň
Czech Republic