

Dear customer!

You have just received a DNA collection kit for performance of a genetic genealogical test.

The kit contains:

- Overview of your order
- 2x sterile brush for cheek swabs
- Specimen envelope
- Air cushion envelope with return address
- Informed consent of the customer to the genetic genealogical testing

Please read carefully the customer's informed consent and complete the form and sign it.

Collect the DNA sample:

1. Wait at least 15 minutes after eating and drinking for sample collection.
2. Take the sterile brush out of the packet. Make sure that the brush does not touch any other surface than the inside of the mouth of the person to be tested.
3. Place the brush between the cheek and gums and rub the buccal inside the mouth about 10 times using an up and down motion as you rotate the swab. Try to make sure that both sides of the brush come into contact with the buccal. After the sample is taken, you should see the collected buccal cells on the brush.
4. Let the brush air dry for short time – please hold it in your hand, do not put it down on any surface. The wet brush may begin to putrefy on the way to the laboratory.
5. Place the dry brush directly into the long specimen envelope marked with the sample number.
6. Repeat the sample collection with the second brush. Place it to the first brush into the same specimen envelope. Close and seal the envelope.
7. Place into the air cushion envelope with the return address:
 - The signed and completed informed consent
 - Specimen envelope with brushes
8. Send the air cushion envelope per mail to: **Genomia s.r.o., Janáčkova 51, 323 00 Plzeň, Czech Republic**

The results will be sent to the address stated in the informed consent within 14 days after receipt of the sample in our laboratory.

Should you have any further questions, do not hesitate to contact us per e-mail laborator@genomia.cz or phone us **+420 3737 49999**.

Best regards

Genomia Team